## Retainer Brite® Dental Appliance Cleaner/Disinfectant

**INTENDED USE:** Product is intended to clean and disinfect a variety of removable dental appliances.

**INDICATIONS FOR USE:** Product is indicated for patients who are using removable dental appliances such as retainers, aligners, dentures, sports mouthguards, nightguards, bruxing and TMJ appliances, which need to be cleaned at home.

**CONTRAINDICATIONS:** This product is contraindicated for patients allergic to Persulfates.

**⚠️ WARNINGS:** Contains persulfates which are a known allergen. Keep out of reach of children. Use only as directed. If product comes into contact with eyes, wash eyes thoroughly with water for several minutes. Do not put tablets or cleaning solution in mouth. If ingested, seek medical assistance immediately. In the USA, please call Poison Control Center at 1-800-222-1222. In the UK, please call NHS Direct at 0845 4647.

**⚠️ PRECAUTIONS:** Do not allow wire appliances to soak for more than 15 minutes. Remove promptly and rinse thoroughly with water. Wash hands thoroughly before and after handling tablets. Do not mix with other cleaners. Single use only. If re-used, product will not perform as intended. Store in a cool dry place.

**⚠️ CAUTIONS:** Keep out of reach of children. Do not put tablets or cleaning solution in mouth.

**ADVERSE REACTIONS:** Allergic reactions.

**STEP BY STEP INSTRUCTIONS:** Use daily for best results.

**1** Drop one tablet into enough warm (not hot!) water to cover appliance.

**3** When blue clears (15-20 minutes\*\*) appliance is clean and odor free!

**2** Place appliance into bubbling blue solution.

**4** Rinse appliance and hands thoroughly with running water.

Kills 99% of seven common odor-causing bacteria including: Staph aureus, E. coli, P. aeruginosa, S. epidermidis and E. hirae. Data on file. Independent 2009 U.S. lab testing (ASTM method E1153).

**\*\*DO NOT allow wire appliances to soak for more than 15 minutes. Remove promptly and rinse thoroughly with water.**

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